



# TIPS FOR THE USAGE OF TEST STRIPS



## wellion<sup>®</sup> vet

Approach the test strip to the blood drop and touch it in a right angle.

Blood is automatically drawn into the test strip.

Let the test strip sip in blood until the white circle in the control window is surrounded with blood - see illustration.

The meter counts down and after 5 seconds your glucose testing result will appear on the meter display screen.



MED TRUST Handelsgesellschaft m.b.H.  
Gewerbepark 10 • A-7221 Marz  
Tel.: (+43)02626/64190 • Fax: DW 77  
e-mail: office@medtrust.at  
www.medtrust.at • www.wellionvet.at



THE MEDICAL SERVICES COMPANY

WELVET715EN-10-2014



**wellion®**  
Your AUSTRIAN brand



**FAST- AND  
LONG EFFECTIVE**

## **INVERT SUGAR SYRUP WITH VANILLA FLAVOUR!**

- Gelatinous consistency can be easily applied on mucous membranes of the mouth
- Is taken up rapidly via the mucous membranes
- Vanilla taste is well accepted by animals



### **SCALE**

The well visibly scale at the side of the tube makes the usage easier. (Content: ca. 40g carbohydrates)



### **RECLOSABLE**

The tube is reclosable with a screw cap.



### **ENERGY SOURCE**

Immediate and longterm effect due to 3 different types of sugar (Glucose, Fructose, Saccharose)



THE MEDICAL SERVICES COMPANY